


Welcome to your Stiltz Homelift

A discreet, attractive and practical accessibility solution



www.stiltzhealthcare.com



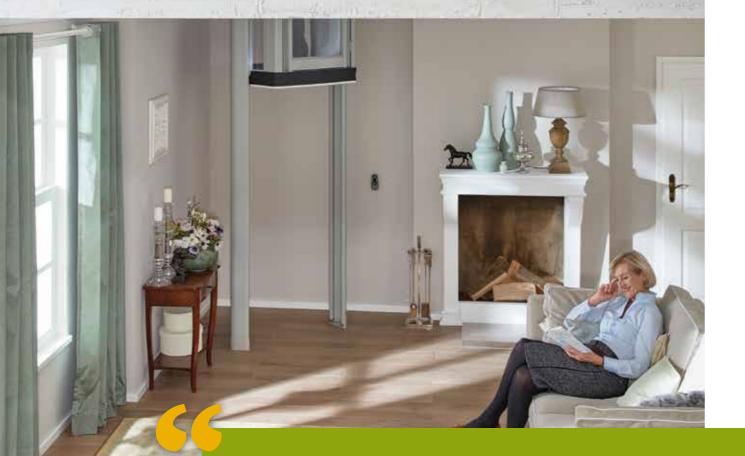

The through floor lift fits neatly into the house and was quickly and neatly installed. It has been reliable and has enabled us to live a 'normal' home life using the first and ground floors. A stairlift would not have done so.

Mrs Goddard via Trustpilot, January 2021

The Stiltz Homelift range. Specially designed to fit easily into your home.

Already popular around the world with many thousands of satisfied customers, the Stiltz Homelift was the first of its kind in the UK.

Designed to make your life easier, a Stiltz Homelift allows you to travel safely and effortlessly between floors, its unique rail and drive system allows you to fit the lift almost anywhere in the home.

Taking up minimal space, and with the smallest footprint, the Duo⁺ range gives you flexibility in where to fit your lift, whilst maximising your living space. The Trio⁺ range gives you the convenience of a larger lift car size if you need to travel in a wheelchair - either now or in the future.

Committed to accessible living, the unique Stiltz Homelift range allows you to remain living independently in your home. For complete peace of mind, a comprehensive 12-month parts and labour warranty is provided with all new installations, with the option of extended service and maintenance plans if desired.

So, if you're looking for an innovative, attractive and useful long-term accessible living solution, the answer has to be a Stiltz Homelift.

In addition to the tremendous honour of being awarded the **2020 Queen's Award for International Trade**, Stiltz are enormously proud of their business, trade and consumer achievements over the last few years:

















A Compact and Elegant Homelift

The Stiltz Duo+ through-floor homelift is designed to make it easier for you to move around the house. Whether you're starting to find the stairs a bit much, or are planning ahead by future-proofing, you will find a Duo+ will fit discreetly into any home. A Duo⁺ is elegantly curved and with its clear car body, it will perfectly compliment your style of décor. It's so versatile that it can be installed in the corner of a room, in a cupboard or in a stairwell void. Alternatively, it can take pride of place in the centre of a room or entrance hallway.

We all need to do our bit for the environment, and a Stiltz Homelift is no different. Each one of our models plugs into a standard mains socket and uses no more electricity than switching your toaster on!

Installation is simple – in the majority of cases work is completed within a single day and once your homelift is fully working, our friendly installation engineers will guide you through the simple usage instructions.





Trio+

Trio+ Spacious Accessibility

The Stiltz Trio+ is a larger homelift designed to comfortably carry a fullsized wheelchair, or three people. The homelift's clear body helps it blend seamlessly into your home and the unique shape means the rails are discreetly recessed behind the slightly wider entrance/exit.

The Stiltz Trio+ has been designed to offer more space, comfort and safety for those who need a larger lift in their home.

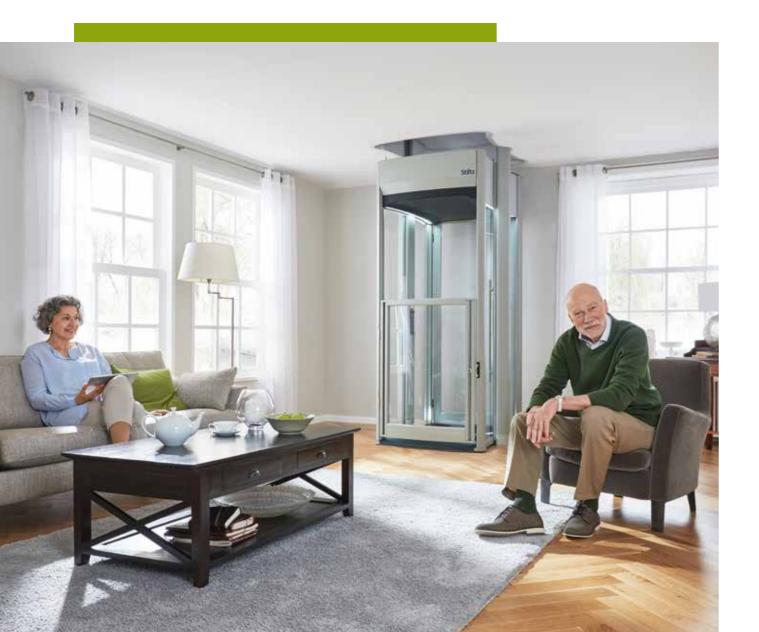

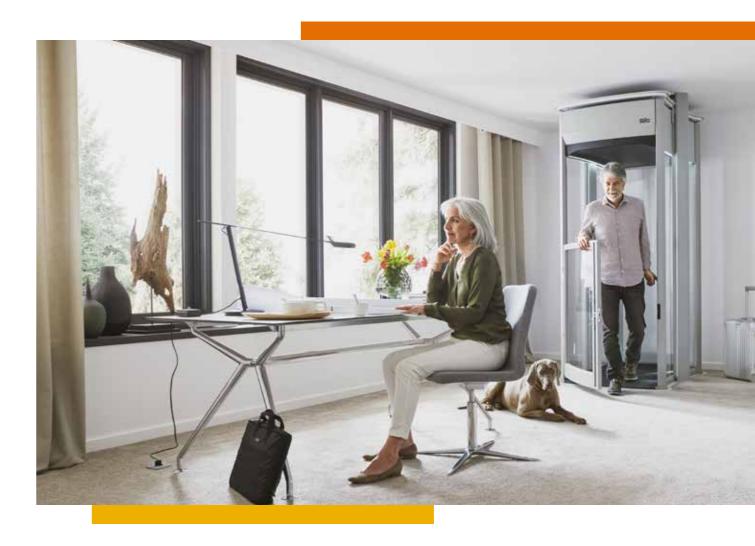

The combination of style and practicality has been perfectly balanced with the Trio+.

Due to its size and weight loading, the homelift can comfortably accommodate a fullsized wheelchair. The gently-angled integral ramp and the wider entrance/exit space helps make access very simple.

Just like the Duo⁺, the Trio⁺ Homelift can be installed quickly without the need for load-bearing walls. It also travels quietly and uses minimal power when moving between floors.

The Trio⁺ is available with a single-door configuration or as a ThruCar, with two doors facing each other. This feature which is unique to Stiltz Homelifts makes entry and exit as straightforward as possible; especially for wheelchair users.

It is important to select the right product for you needs - now and in the future. To ensure compatibility, Stiltz always recommends discussing current or future wheelchair dimensions with our trusted assessors when they visit. My lift has made such a difference. It means I can stay in my home with no worries about having to move as my disabilities increase. The whole project from start to finish was excellent and well managed. The product is very good and in my opinion the service is second to none. Everybody I have dealt with within the company has been very professional and helpful. I would recommend Stiltz to anybody.


Mrs. Topley via Trustpilot, February 2021



Stiltz

Extended-travel Trio+ Homelift

In addition to the standard 4.5 metre travel offered by both Duo+ and Trio+, the Stiltz Trio+ Homelift is now available with 6 metre travel allowing it to be fitted in homes with very high ceilings or conversions. It's also ideal for homeowners looking for a 3-stop homelift.

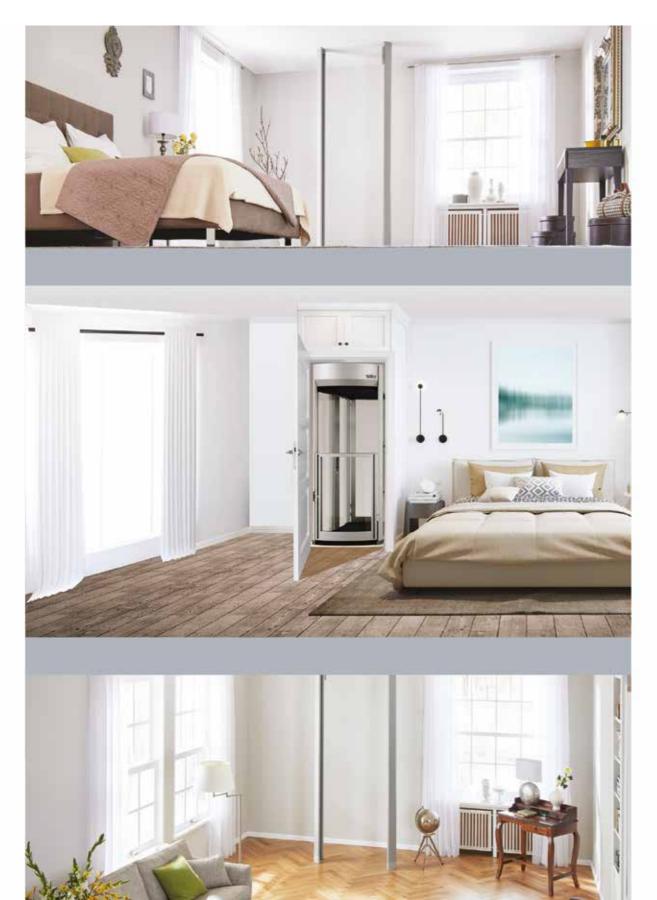

Why Choose a Stiltz Homelift?



Maximise your living space Move your lift to another floor and with only two slim rails remaining, you'll hardly know it is there.

Small footprints

Both the Duo+ and the Trio+ have extremely compact footprints, despite the fact that the Trio+ will accommodate a full-sized wheelchair.



Packed with safety features

Multiple sensors are fitted to every Stiltz Homelift to ensure that everyone remains safe – both inside and outside the lift.



In-car control panel

The in-car control panel is designed to be extremely easy to use with push-button, hold-to-run controls.





Elegant interior lighting

Subtle and attractive internal lighting at the top and bottom of the lift car means you can travel in safety any time during the day or night.



Remote control

Your homelift comes with two remote control handsets included. Additional handsets can be purchased if required.



DuoClassic

Our Entry-Level, Robust Homelift

The Duo Classic - the original Stiltz Homelift.

The Duo Classic is our entry-level product, it has solid sides and like the rest of the Classic range it is available in a clean, white finish. Dependable, affordable and with all the same great features as our more highly-styled products.


DuoThruCar Unique Dual Rail Technology

Enter and exit through opposite sides of the homelift car

The Stiltz Duo ThruCar allows users to enter and exit from both sides of the car.

Configuring the lift car with two doors which face each other is only made possible by the dual rail technology which is unique to Stiltz. Because the lift car runs on rails positioned at the sides, this allows unimpeded access to both front and back of the car.

There is no other domestic lift on the market that is capable of having a double entry or ThruCar configuration. It offers all of the same features as the Duo Classic.

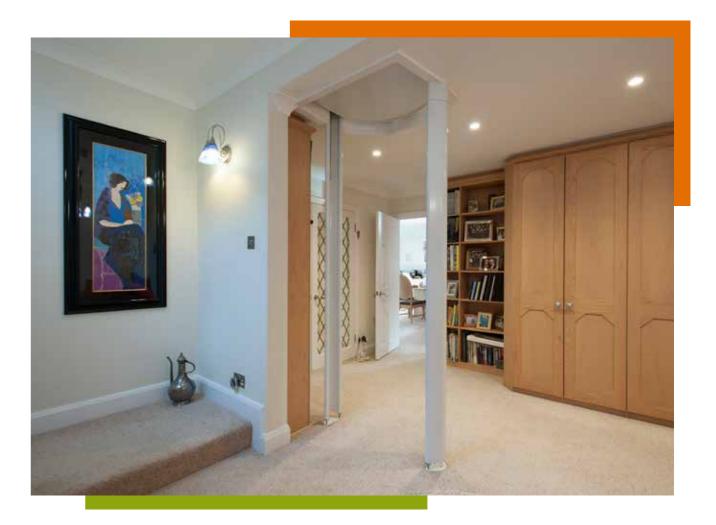

Stiltz Homelifts Features & Benefits



Small Footprint

The Duo⁺ footprint is a remarkably small 0.55m². Still surprisingly compact, the Trio⁺ has a slightly larger footprint of 1.25m² and is designed to accommodate a full-sized wheelchair. Please discuss current or future wheelchair dimensions with our trusted assessors when they visit.



Quiet Travel

Powered by a self-contained, electric drive motor which is completely hidden away in the lift car roofspace. There are no noisy hydraulics with a Stiltz Homelift.



Fast Installation

Standard installations can take as little as one day once the ceiling aperture has been created by a qualified builder.



Economical to Use

A Stiltz Homelift plugs into a single, standard 13-amp domestic power socket - just like any other domestic appliance. When running, it uses no more power than if you were to switch your toaster on.



Unique Dual Rail System

It's what makes a Stiltz Homelift so wonderfully versatile; our self-supporting structure of two parallel, vertical rails bear the entire load of the lift. No additional burden is placed on the structure of your home.



Compact Drive System

Our unique, self-contained drive system is cleverly concealed in a single space within the roof area of the lift car. No need for additional bulky plant rooms or other machinery.



Automatic Door (Trio+ Model)

To make entry and exit more straightforward for wheelchair users or customers with limited mobility, the Trio⁺ has an automatic door available as an optional extra. The maximum opening angle of the powered door is 90 degrees.



Wide Entry/Exit (Trio+ Model)

Wheelchair users will find the wide entry and exit points of the new Trio⁺ particularly useful when manoeuvring their chair in confined spaces.



Easy Access (Trio+ Model)

For safety and comfort, the Trio⁺ comes with a gently-angled ramp to enable more convenient entry for wheelchair users and those unable to manage steps. The Duo range has a 100mm step at ground floor level, with level access on the upper floor. Please speak with our trusted assesor or surveyor during your appointment as they will be able to advise further as to the most suitable product for you and your home.


Typical Lift Installation Applications







Standard

This is our most popular configuration, the homelift will travel between a downstairs room such as living room and arrive upstairs, typically, in a bedroom.

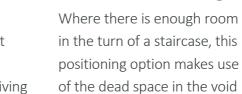

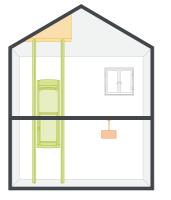

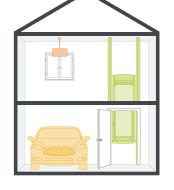

Sloping/vaulted ceilings


Our vertical rails need to fix to a horizontal surface. But don't worry if you have a vaulted ceiling. Our building teams can provide subtle solutions to overcoming pitched roof installations.

Garage If you are looking to get from your garage safely into your home we can build a simple shaft around your Stiltz Homelift to take into account health and safety considerations.




Stairwell void/landing

making the homelift even more unobtrusive.

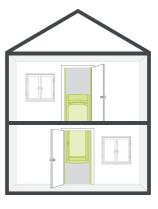

Cupboard

Thanks to the compact nature of a Stiltz Homelift. you can choose to have your lift completely hidden by installing it inside an airing cupboard or wardrobe.

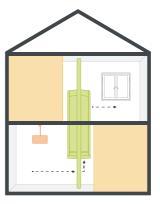

ThruCar

Unique to Stiltz Homelifts is the ThruCar concept. Enter and exit through opposite sides of the lift car - ideal for wheelchair users or for lifts built into a stairwell void.



FAQs

Is a Stiltz Homelift suitable for wheelchair users?

Yes. The Stiltz Trio⁺ Homelift is ideal for homeowners with mobility issues as it allows them the freedom and independence to move around the home unhindered.

The Trio⁺ is our widest and deepest model and can accommodate a full-sized wheelchair. It is an ideal alternative to a stairlift and perfect if you have limited mobility. Stiltz always recommends discussing current or future wheelchair dimensions with our trusted assessors when they visit.

The Duo range of homelifts normally have a step of 100mm at ground floor level, with level access on the upper floor. Again, our trusted assessor will be able to advise further as to the most suitable product for you and your home.

What happens if the power fails and I am in my Stiltz Homelift?

In the event of a power cut to the home, all Stiltz Homelifts are equipped with a battery back-up facility. This is included as standard with the lift and will allow anyone in the lift at the time of power failure to gently descend to the lower level of the home, exit the lift, and wait until the power comes back on. If you live in an area prone to power outages, Stiltz can provide the optional extra of an uninterruptible power supply. This will enable your homelift to continue to be used several times for the duration of a power cut.

Does the Stiltz Homelift have a door?

All Stiltz Homelifts are fitted with either one or two hinged doors depending on whether you order a ThruCar configuration (selected models only). Our standard door is a half height door. The doors will lock automatically during lift travel, and the lock will release once the lift comes to a stop and you are ready to exit the car.

Can I customise my Stiltz Homelift?

The Stiltz Homelift has a range of options available such as grab-handle and seat. Colour customisation of car and rails is available on selected models within the Classic range.

What building work will need to be carried out?

Unlike conventional lifts, all that a Stiltz Homelift requires for installation is the appropriate aperture to be cut out and trimmed by a qualified builder. The work itself is minimal and not time consuming. There may be instances where small amounts of additional work may need to be carried out to enable your homelift to be fitted exactly where you want it to go. These works can include repositioning of a wall, moving pipes and electrical cabling. The work will be managed by our building teams and our surveyor will discuss this with you during their visit.

How safe is the Stiltz Homelift?

The Stiltz Homelift has safety sensors fitted above and below the lift car which detect obstructions and smoothly stop the car. The lift is also fitted with a full height light curtain which acts as a sensor and ensures nothing crosses into or out of the lift as it travels. The light curtain ensures the lift will come to a steady stop until you are ready to continue your journey. Stiltz engineers are on call 24/7 in the event of a lift emergency.


Is VAT applicable on a Stiltz Homelift?

VAT is chargeable on all Stiltz Homelifts but if you are disabled or have a long-term illness you may be exempt from paying VAT on products designed or adapted for your own personal or domestic use. If you think you may qualify for VAT relief, please request an exemption form from us via the phone in advance, or from your trusted assessor or surveyor when the attend your property. VAT is chargeable on all Stiltz Homelifts but if you are disabled or have a long-term illness you may be exempt from paying VAT on products designed or adapted for your own personal or domestic use.


If you think you may qualify for VAT relief, please request an exemption form from us via the phone in advance, or from your on site surveyor when they attend your property.

Lift Footprints

Duo+ Weight limit 375lb / 170kg

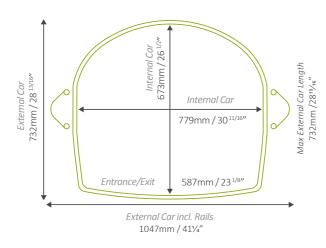



Duo ThruCar Weight limit 375lb / 170kg

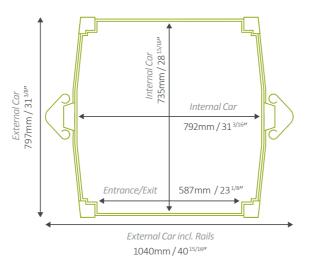

Trio+ Weight limit 550lb / 250kg

MAX. EXTERNAL CAR WIDTH 967mm / 381⁄16"

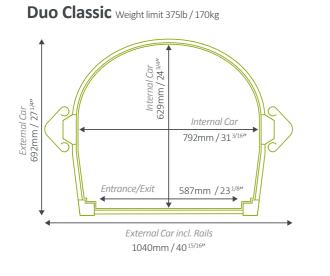

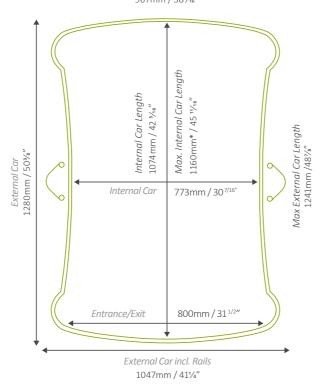


The Trio+ is designed for wheelchair usage. However, to ensure that the product you are considering is suitable, please speak with your Stiltz trusted assessor as wheelchair sizes can vary and it is important for us that you choose the correct lift for your needs.

Please note that these dimensions are subject to change, for guidance only and should not be relied upon for construction purposes.

To allow for height limitations on the upper floor, all Stiltz lift cars can be supplied with a shorter cabin if needed. Your on site surveyor will explain more about these special orders if needed.

The Duo range has a 100mm step at ground floor level, with level access on the upper floor. Please speak with our on-site surveyor during your appointment as they will be able to advise further as to the most suitable product for you and your home.

* Please note that this measurement is taken between the polycarbonate panels

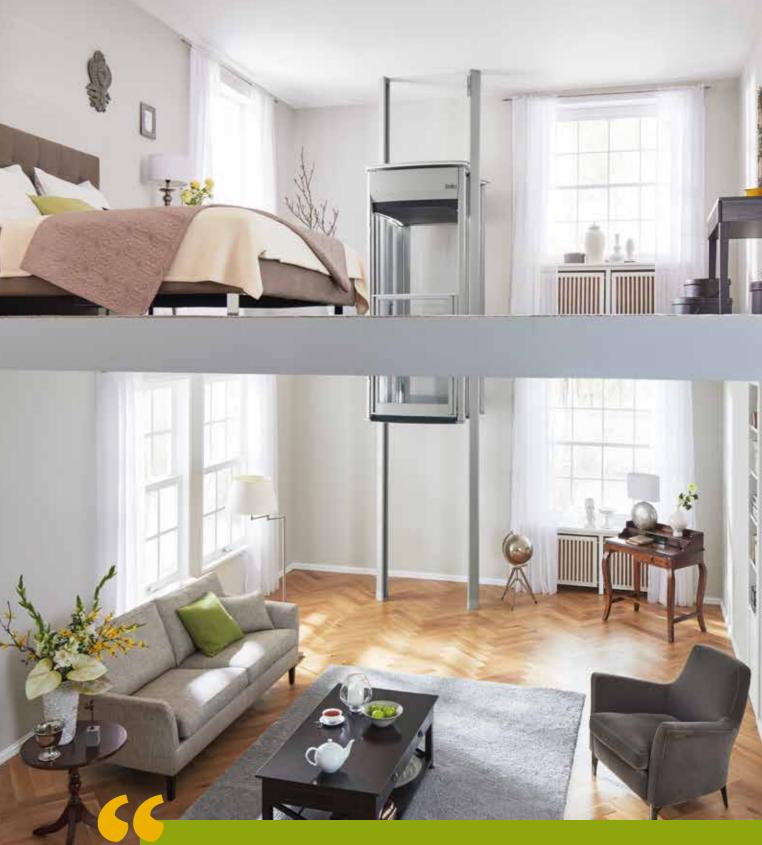

We We frie Still as I thir tha

We have had a Stiltz Lift for 4 years and haven't had any problems with it at all. We have had the annual service each year. We have recommended the lift to friends and several people have been to see it and have gone on to invest in a Stiltz Lift. It is very reassuring to know that we can stay in the family home for as long as we want to and use all the rooms easily. It is also very useful to carry things upstairs and bring heavy items like suitcases downstairs. A great buy, thank you. Mrs. Rowe via Trustpilot, February 2021

Your approved Stiltz Trade Partner

Stiltz Ltd.

Building E, Prime Point Dandy Bank Road Pensnett Estate Kingswinford West Midlands DY6 7TD

01384 880343 enquiries@stiltz.co.uk www.stiltzhealthcare.com

Stiltz Homelifts are available throughout the UK.

